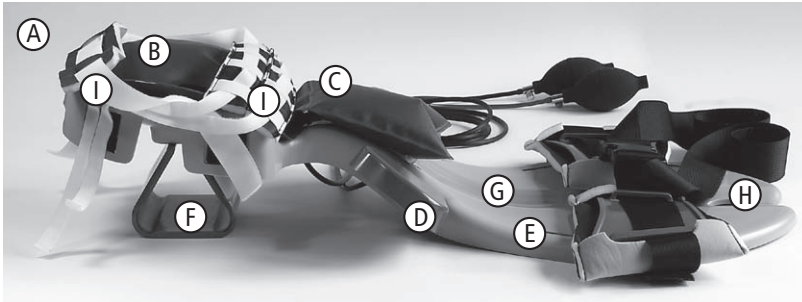
# BEACH-CHAIR+

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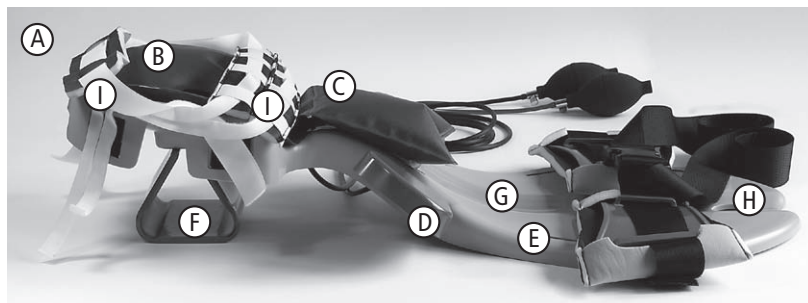
## BEACH-CHAIR<sup>+</sup> device prepare for the operation:

- Wipe disinfection where necessary (use no chlorine!)
- Prepare cushioning material (set) for patient
- Function tests: No defects on device, inflatable cushion, storage bag, fastening tapes, wings
- Material: Special synthetic material, X-ray permeable

## Legende

- A. BEACH-CHAIR<sup>+</sup> device
- B. Inflatable pillow for head (black tube)
- C. Inflatable cushion for neck (red tube)
- D. Detachable wings on both sides
- E. Thorax plates with straps
- F. Support block for head rest
- G. Depression for spinal column
- H. Hole to hang up BEACH-CHAIR<sup>+</sup> device
- I. Disposable straps to hold chin and forehead in position
  - Storage bag (not shown)

# Scope of supply

---

## Consumables (set)

For one-off use on patient.

1. Forehead cushions, no latex
2. Chin cushions, no latex

3. Thorax cushions (2 in number), no latex

PE storage bag, no latex

4. Velcro fastening tape for use 5 times

5. Pillow for supporting head (used in case of degenerative changes in the spinal column)

## Option

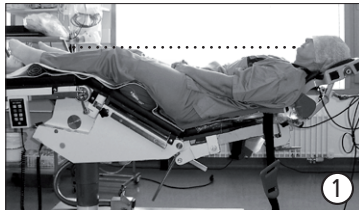
6. Face protector

## Preparation

### Important positioning information:

The storage bag is vital as it allows the beach chair device to slide (extend).

To minimise any slipping when positioning the patient and laying them back, it is better to induce the anaesthesia already when in a gentle beach chair position, i.e. the **seat plate** must be brought into the maximum **Trendelenburg position** and the **back plate** brought into a slightly **raised position**. (Figure 1 with line from tip of chin to patella)

1

### Preparation of beach chair device

1. Insert beach chair device (abdominal part) into the **storage bag**.
2. **Pull thorax belts (E)** out of the storage bag through the side openings.
3. Provide some cushioning for the **thorax plates**.
4. Press together the **support block (F)** for the head rest and remove (Figure 2).
5. Release air from the **pillow and neck cushion (B+C)**.
6. Insert the straps through the **chin and forehead cushions**.
7. Secure **straps** to head rest (Velcro fastener).

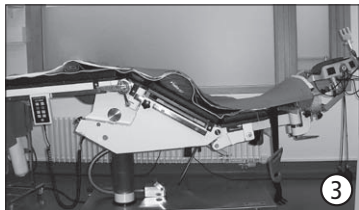
2

### Preparing operating table

1. Align in the usual starting position or 'gentle' Beach Chair position (Figure 3).
2. Remove cushions from head rest.
3. Beach Chair device into the head rest.
4. Adjust height of head rest (abdominal part of Beach Chair device must lie flat on the operating table. ).
5. Align the head rest with the Beach Chair device inserted to small patients.
6. Let the abdominal belts hang from the sides of the operating table.
7. Cover the operating table and Beach Chair device with the usual cover as far as the shoulder wings.

3

# Positioning

## Positioning patient (before the anaesthetic)

1. Fold down the **wings (D)** and hook in position (on the side from which the patient gets onto the operating table, Figure 4).
2. **Position the patient's pelvis at the lowest point of the operating table.**
3. Head rest holders and pull out together with the Beach Chair device (**do not adjust height**).
4. **Neck cushion** at the patient's neck.
5. Tighten the holders.
6. **Pillow and neck cushion** until the head and neck are lying on a soft support.
7. Reinsert the folded-down **wings** (stabilises the shoulders).
8. When secured with the **thorax belts**, the patient can easily be brought into the operating theatre.

## Ventilation and intubation

**Important:** After the patient has been prepared for the anaesthetic and the latter has been administered, there are now two further measures to facilitate ventilation and intubation.

**Thesemeasures should always be coordinated with the anaesthetist.**

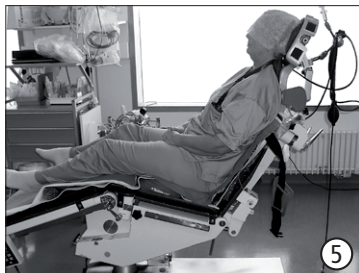
1. Release air from the pillow.
2. Add air to neck cushion (the head is slightly reclined).

Ventilation and intubation is carried out. If necessary, the patient's **head** can be raised by adding air to the pillow and placing the **blue support pillow underneath**.

### After intubation

3. **Release air** from both cushions.

### Positioning patient (for the operation)

1. **Seat plate** into the maximum Trendelenburg position
2. Patient into the required position (Figure 5). If the patient is to sit in an even more upright position, reverse the Trendelenburg position.
3. Add **some air to the pillow** until the patient's head is at the required position.
4. For additional support, add some air to the **neck cushion**.
5. Using the **chin and forehead straps**, secure the head in position (adequately but not tightly).
6. Align both **thorax belts, plates must rest against the sides of the rib section of the thorax, below the nipples**.
7. **Close** the thorax belts using the fastener.
8. **Tighten** the thorax belts **simultaneously** (fastener must be in the middle).
9. **Ask anaesthetist about ventilation pressure**.
10. **Fold down wings** on the operating side and hook in position. The patient is then **disinfected and covered with a sterile cover**.
11. If the wing is required during open surgical procedures, reinsert it.

### After the operation, return to the original position

**Important:** To prevent the patient «slipping out of the device», we recommend the following measures when **laying back**.

1. **Lower the seat plate** to the lowest Trendelenburg position.
2. **Undo the forehead and chin straps** (Figure 6).
3. **Lower the back plate** to the required height.
4. If required, i.e. if the head is slipping slightly from the head rest, add some air to the **neck cushion** until the head is lying on a soft support.
5. Use **forehead straps** to secure the patient's forehead lightly in order **stabilise the head**.
6. **Thorax belts** should only be undone **before** transferring the patient to another bed.

## Scalene block

The Beach Chair device is used to stabilise the patient's neck or spinal column. In order to ensure better anatomical support, air can be let out of the neck cushion as required (the head is reclined). If the wing is now folded down, the shoulder falls back slightly, allowing better access to the scalene nerve. After positioning the scalene block, let air in or out of the two cushions again until the patient is lying comfortably.

## Clavicular fractures

In case of «clavicular osteosynthesis» the patient's head can get in the way. If agreed with the surgeon, air can be let out of the pillow and more air added to the neck cushion. This allows the head to be reclined. Access is better if the head is turned to the side. In any case, after the «osteosynthesis», the previous status must be restored.

## Cleaning

Wipe disinfection with normal products such as Ecolab, Samaclean etc. Use no desinfectant which contains chlorine!

## Disposal of Beach Chair device

- disposal in a PE bag and by incineration or
- return to Samarit or the distributor at the user's own cost. The return shipment must be accepted in advance.
- With new purchases, return and disposal are free of charge.

## Warranty

2 years (condition to follow operating instructions)

## Technical Data

Storage temperature:

Room temperature

Dimensions L x W x H:

800 x 320 x 170 mm

Material: Special synthetic material, X-ray permeable, latex-free, physiologically safe, impact-resistant, closed pore

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We reserve the right to modifications as a result of continuous adaptation and further development.

**SAMARIT**  
improving patient care

**Samarit Medical AG**  
Grundstrasse 14 / PO Box 213  
CH-8126 Zumikon, Switzerland  
Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39  
e-mail: [info@samarit.com](mailto:info@samarit.com) - [www.samarit.com](http://www.samarit.com)