

# SAMARIT

safe patient moving & handling



## SAMARIT "no lift" Transfer Systems

Rollbord™ **HIGHTEC**

For  
**HEAVY**  
use!

### The safe way to transfer patients

A legal requirement in some countries, taken for granted in others – working conditions should be as comfortable as possible. Healthcare workers should avoid lifting when transferring patients. **SAMARIT Products** assist with transfer, without the need to lift. The products are designed to facilitate the transfer without using a lot of effort, thus making the work easier for the carers and kinder to their backs.

**STANDARD**

*foldable*



\*SWL: 240 kg  
\*SWL: 529 lbs

Suitable for radiology, operating theatre, intensive care units and all special wards.

**Size:** 178 x 50 cm / 89 x 50 cm, 2.7 kg  
70 x 20 inch / 35 x 20 inch foldable

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

**Order No. 440.0100**  
UDI/GTIN: 7640144460013

**ICU-BARIATRIC**

*non-foldable*



\*SWL: 310 kg  
\*SWL: 684 lbs

Suitable for Intensive Care (ICU) nursing services, on air and decubitus mattresses.

**Size:** 178 x 50 cm, 3.8 kg  
70 x 20 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

**Order No. 440.0200**  
UDI/GTIN: 7640144460020

**MINI-GYN**

*non-foldable*



\*SWL: 180 kg  
\*SWL: 397 lbs

Suitable for urology operating theatres, gynaecology operating theatres, wake-up stations and emergencies.

**Size:** 91 x 50 cm, 1.4 kg  
36 x 20 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

**Order No. 440.0300**  
UDI/GTIN: 7640144460037

\*SWL = Safe Weight Load

## 5 different models to suit every hospital application

- High performance
- Radioluscent
- Non-toxic materials
- Anti-bacterially surface
- Safe and comfortable for patients
- Prevents back injuries to care workers
- Skin-friendly (no latex)
- Easy to clean and disinfect – hygiene approved
- Disposable covers available
- Tear resistant nylon cloth
- Bridges gaps up to 20 cm
- Lightweight, easy suspendable

**COVER DISPENSER**

*Suitable for all Rollbord models*



Wall mount

**Size:** 15 x 66 x 14 cm  
0.6 x 2.6 x 0.55 inch

**Order No. 440.1088**  
UDI/GTIN: 7640144460495

**SURGIBOARD**

*non-foldable*



\*SWL: 180 kg  
\*SWL: 397 lbs

Suitable for operating theatre, X-Ray, various wards (replaces aluminium rollers).

Size: 120 x 40 cm, 1.5 kg  
47 x 16 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0400  
UDI/GTIN: 7640144460044

**MRI/CT**

*foldable*




\*SWL: 200 kg  
\*SWL: 440 lbs

Suitable for transfer of injured patients on/ from stretcher.

Size: 178 x 40 cm / 89 x 40 cm, 2.5 kg  
70 x 16 inch / 35 x 16 inch foldable

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0500  
UDI/GTIN: 7640144460051

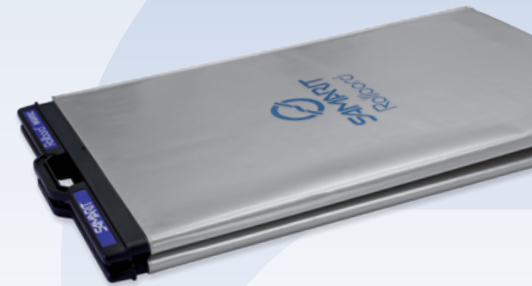

**DISPOSABLE COVER**

*Suitable for all Rollbord models*



To protect Rollbords from blood and dirt. Size:

200 cm, Ø 60 cm (50 pcs) 79 inch, Ø 24 inch  
135 cm, Ø 60 cm (74 pcs) 53 inch, Ø 24 inch

200 cm / 79 inch Order No. 440.1000  
UDI 7640144460471

135 cm / 53 inch Order No. 440.1400  
UDI 7640144460488



# SAMARIT "no lift" Transfer Systems make day-to-day work with patients easier

**SAMARIT also gives comfort and security to patients**

In hospitals, where patients may need to be transferred carefully several times a day, **SAMARIT** makes life easier for the carer and more comfortable for patients.

**SAMARIT Transfer Systems – a great help to healthcare professionals – worldwide!**

If you imagine how many transfers are carried out every day in hospitals and within communities then you will appreciate the true value of the original **SAMARIT** systems. In fact, over 150'000 **SAMARIT** Transfer Systems are being used by nursing professionals working worldwide.

All of them find that their work is made easier when using **SAMARIT** products.

different model weights 1.4 - 3 kg and can be set up quickly – anywhere



The safe and secure way to transfer patients



Turn the patient onto the side, slide the board under the body and return the patient to the original position.



Gently push the patient at the shoulder and hip, without applying force, and transfer.



Turn the patient onto the side. Remove the Rollbord starting at the feet.

### Environmentally friendly

**material\*** SAMARIT transfer aids are made of ecologically safe materials. They can be disposed of in an environmentally friendly manner and if a fire should occur they do not produce toxic fumes.

\* Teflonised sliding fabric around a polyethylene foam core, coated polyester cloth, PE security endings.

### Easy cleaning

The Rollbord can be cleaned best with *SamaClean Wipes* or any hospital disinfectant. For anticipated soiled conditions use the Rollbord disposable cover. It covers ends of Rollbord and allows any residue to be folded untouched into the cover, leaving a clean surface. Repairs can only be carried out by authorized personnel (warranty).

### High quality standards

SAMARIT transfer aids have:

- CE mark
- Design patents

EU-SRN: CH-MF-000016280  
CHRN: CHRN-MF-20000184

[www.samarit.com](http://www.samarit.com)

Ask for separate brochures for the other products in the SAMARIT range!



SAMARIT Rollbord ECOLite



SAMARIT Rollbord H-Line



SAMARIT Transglide



SAMARIT Nursing Aids

Download your language from our website or ask your local distributor.



**Samarit Medical AG**

Gewerbestrasse 12  
CH-8132 Egg, Schweiz

Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39

e-mail: [info@samarit.com](mailto:info@samarit.com) - [www.samarit.com](http://www.samarit.com)

